Helping you Wear it Right

Putting On Your 3M™ Particulate Respirator 8670F*

For Use by the General Public in Public Health Medical Emergencies.

For proper use, see the *User Instructions* provided inside product packaging.

1 Prepare To Use Respirator

- Wash your hands thoroughly with soap and water before handling.
- Unfold the respirator by pulling top part (with foam) up and bottom part (with the words printed on it) down so the respirator opens all the way.
- Inspect respirator for holes, tears or other damage to surface, straps, and nose foam. Get a new respirator if damaged.
- Remember, nothing should be between your face and the respirator, including hair, facial hair, jewelry, clothing, etc.
- If you wear glasses, read steps 2 through 4, then remove glasses before putting on the respirator.

2 Place Respirator On Face

- Place the respirator against your face, with the bottom under your chin and nose foam across bridge of your nose.

- Pull the top elastic band over your head and place high at the top back of your head.

- Next, pull the bottom elastic band over your head and place around the neck below the ears.

3 Adjust The Metal Nose Strip

- Use both hands to bend the metal nose strip to fit snugly against your nose and face. The respirator may not fit as well if you pinch the metal nose strip using one hand. Use two hands.
- Slide fingers down both sides of metal nose strip to seal it against your nose and face.

4 Check The Respirator-To-Face Seal

Remember: Putting the respirator on correctly means more of the air that you breathe goes through the respirator filter.

Completely cover the middle panel of the respirator with one or both hands. Do not push the respirator against your face. With your hands in place on the surface of the respirator,

exhale or breathe out sharply. If you feel air blowing on your face or eyes, the respirator needs to be adjusted. To adjust, repeat steps 2,3 and 4. When the respirator has a good fit, you will not feel any air blowing on your face or eyes. If you can't get a good fit, try a different model respirator. Return glasses to face, if applicable.

Although the 8670F respirator is designed to fit a variety of adult face sizes, it is not designed for use by children.

How To Remove And Dispose Of Your Respirator

Go to a clean, safe area away from other people to remove your respirator.

Important: Touch only the straps and not the surface of the respirator when removing. It is recommended to wash your hands thoroughly with soap and water or an alcohol-based hand cleaner before and after removing your respirator.

- Without touching the respirator, lift the bottom strap from around your neck up over your head.

- To avoid snapping the respirator, SLOWLY lift off the top strap.

- Properly dispose of your used respirator by carefully placing in a closed waste container. You should never share respirators. Wash your hands after disposal.

*These instructions have been specifically designed for model 8670F, a non-valve, three-panel folded respirator for non-occupational use. Fitting instructions for other respirators can be found at www.my3MN95.com.

⚠ WARNING

IMPORTANT: This respirator is intended to help reduce breathing in pathogenic biological airborne particulates or germs during public health medical emergencies, such as an influenza pandemic. If you do not follow all instructions and limitations on the use of this respirator and/or do not wear this respirator during all times of exposure to the airborne germs, the respirator may not be as effective. The respirator cannot stop the breathing in of all germs in the air and does not eliminate the risk of disease or illness. For proper use of this respirator, see the *User Instructions* inside the package or call 3M at 1-888-436-3636.

3M General Offices

3M Center
St. Paul, MN 55144-1000
www.3M.com

Please recycle. Printed in USA.
© 3M 2008. All rights reserved.
70-0714-8830-1

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